# Dream Belt

Réfs. : A062000 - A062010

Notice d'utilisation • Instructions for use  
Benutzungsanleitung • Gebruikshandleiding  
Instrucciones de uso • Instruções de uso  
Istruzioni per l'uso • Návod k použití  
Инструкция по использованию • Használati utasítás  
Instrucțiuni de utilizare • Instrukcja użytkowania  
Brugsanvisning • تعليمات الاستخدام


Babymoov  
Parc Industriel des Gravanches  
16, rue Jacqueline Auriol  
63051 Clermont-Ferrand  
France  
www.babymoov.com

**Hotline (only for France) :**

**04 73 28 37 37**

Designed and engineered  
by Babymoov in France

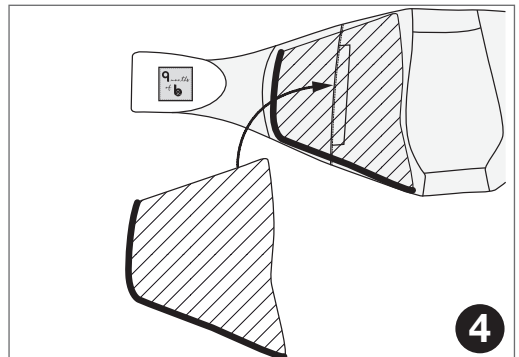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### Advice on using and installing the belt

The Dream Belt for sleeping is worn essentially lying stretched out on the side.

- To install the belt stand up and place it directly on your belly (**picture n°1**).
- To attach the belt use the adjustable Velcro® on the back. The label « 9 months of b » on the back must be positioned so as to be read from left to right (**picture n°2**).
- To position the belt correctly check it is properly centred on your belly (**picture n°3**).
- Adjust your belt if necessary depending on your stage of pregnancy. It is up to you find how tight to pull your belt to feel supported and comfortable.
- The belt can be worn right next to the skin or over clothing.

### Maintenance tips

- Before washing the Dream Belt remove the memory foam from each pocket.
  - Machine-wash the belt at 40 °C (delicate cycle) to preserve the colours. Washing at 60 °C is possible.
- NB:** close the belt well with the Velcro® so as not to damage it during the wash.
- Once the wash is over dry it flat well away from any heat source.
  - To put each piece of foam back in its pocket correctly turn the belt over so that the inner side is flat facing you. Make sure the label « 9 months of b » is readable to your left (**picture n°4**).
  - Once the inner side is positioned correctly in front of you direct the foam's longest point downwards to fit it into the corresponding pocket (**picture n°4**).

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